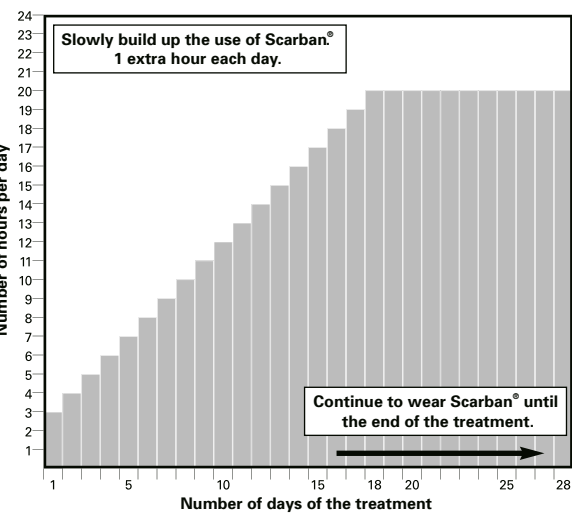
Scarban®

Scarban® scar care products

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EN

Scarban® – a set of high-quality products for the treatment of scars.

Indications

- Scarban® is recommended for the treatment of new and existing keloid and hypertrophic scars.

Scarban® product description

- Self-adhesive silicone bandages for scar treatment; available in a Light version and in an Elastic version.
- Scarban® silicone bandages are made of 100% medical silicone gel, an elastic layer and a protective layer.
- Medical silicone garments and special silicone bandages, to be cut to size, for burns, (plastic) surgery and dermatology. The silicone garments are made of 100% medical silicone gel and an elastic polyamide material.
- Scarban® Silicone Cleaning Soap, a cleansing agent for the Scarban® silicone bandages.
- Scarban® Skin Lotion, a comfort enhancing emulsion for the skin, with caring properties.

Action

Scarban® silicone bandages make scars more supple and reduce their redness. In addition, Scarban® is an ideal bandage for preventing rubbing directly on the skin. The Scarban® silicone bandage is non sterile, adhesive, washable, and suitable for multiple uses. Not to be used on open wounds.

Directions for using the Scarban® silicone bandage

- Cut the Scarban® silicone bandage to the desired shape and size. Cut in rounded shapes without corners (fig. 1). The best result is obtained when the scar is completely covered, and the bandage overlaps the healthy skin by about 2 cm.
- Clean the scar and the skin well, making sure they are dry and free of oils (fig.2).
- Pull off the protective foil from the Scarban® silicone bandage (fig. 3).
- Place the self-adhesive side of the Scarban® silicone bandage on the scar (fig. 4). Apply the bandage gradually as shown in the picture.
- You can secure the Scarban® silicone bandage more firmly with adhesive tape or an elastic bandage (fig. 5).
- When removing the Scarban® silicone bandage from the skin, be careful not to remove the silicone bandage from the same place every time.
- After removing the Scarban® silicone bandage, you can rub the covered surface of the skin with a hydrating cream, such as Scarban® Skin Lotion.

How best to use the Scarban® silicone bandage?

- At the beginning of the therapy, use the Scarban® silicone bandage for 3 hours per day. In a normal course of treatment (that is, no skin irritation), lengthen the period of use by one hour per day until you are wearing the Scarban® silicone bandage a maximum of 20 hours per day. Building up the wearing time gradually like this lets your skin become accustomed to the effect of the silicone bandage.
- For an optimal result, the Scarban® silicone bandage should be worn a minimum of 8 to a maximum of 20 hours per day.

Directions for using the Scarban® silicone glove

- The Scarban® silicone glove is worn if there is scar formation on the hand or on the fingers. The action of the silicone glove makes the skin more supple and promotes the mobility of the joints.
- To pull the glove on, first fold back the edge of the glove.
- Put some Scarban® Skin Lotion on the hand and in the glove and pull it on. Scarban® Skin Lotion also promotes the durability of the silicone layer.
- In any case, never pull the glove on without using a lubricant.
- After pulling the glove on, turn the edge of the glove back into place.

Maintaining the Scarban® silicone bandage

- Rub a few drops of Scarban® Silicone Cleaning Soap into the silicone bandage (fig. 6). The adhesive strength wears off after contact with the Cleaning Soap or water. Rub off all dirt as much as possible.
- Then rinse thoroughly under warm, running water (fig. 7).
- The silicone bandage curls up in contact with water (fig. 8).
- To dry it, lay the (curled up) silicone bandage on a dry tea-towel with the adhesive side face up.
- When it has dried completely, the silicone bandage uncurls by itself and the adhesive strength returns.
- Use the silicone bandage only when it is completely dry.

Duration of usage

When used according to the instructions, the Scarban® silicone bandage Light will last from 2 to 4 weeks, and the Scarban® silicone bandage Elastic from 4 to 8 weeks.

Precautions

- NEVER use the Scarban® silicone bandage on an open wound!
- Scarban® Skin Lotion and Scarban® Silicone Cleaning Soap are specially developed for further enhancing the comfort of the treatment. NEVER use other salves, creams, gels or lotions in combination with the Scarban® silicone bandage, unless this is explicitly prescribed by your doctor.
- Always consult your doctor concerning skin irritation or skin problems.
- Scarban® products are for external use only.
- Scarban® products are produced to exacting quality standards. If the Scarban® silicone bandage should still lose silicone through wear-and-tear at the edges, you can trim the edges by sticking the silicone bandage on a piece of plastic foil and then trimming the edges.
- Never dry the Scarban® silicone bandage in the sun or near a source of heat.
- Scarban® products are entirely safe for treating children's scars. Nevertheless, keep Scarban® products out of the reach of children.

This package contains

1 or 2 Scarban® silicone bandages and 50 ml Scarban® Silicone Cleaning Soap (see package).

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