

# Spincare Treatment Protocol

## Superficial to partial thickness wounds

Option 1\*:

Day 0    Clean the wound site according to standard of care  
          Apply Spincare – and leave without covering it

Next      Patients can shower after 24-48h after Spincare application  
            Spincare matrix peels off spontaneously when wound is healed underneath

\* Feedback from clinical staff that this option often yields the best results

# Spincare Treatment Protocol

## Superficial to partial thickness wounds

Option 2\*:

Day 0      Clean the wound site according to standard of care  
              Apply Spincare  
              Cover with non-sticking layer + absorption

Patients can shower after 24-48h after Spincare application

Day 3      Secondary cover is no longer needed – leave Spincare matrix uncovered

Next      Spincare matrix peels off spontaneously

# Spincare Treatment Protocol

## Superficial to partial thickness wounds

Option 3\*:

Day 0: Clean the wound site according to standard of care  
Cover as per local practise

After 24-48 hours: Apply Spincare – leave uncovered

Next Patients can shower after 24-48h after Spincare application  
Spincare matrix peels off spontaneously when wound is healed underneath

\* If clinicians expect huge amounts of exudate

## Handling the Spincare

- ✓ Keep the Spincare in its docking station for overnight charging
- ✓ Make sure the tip of Spincare device is clean – with no residues of solution
- ✓ Load the Spincare ampule immediately before use
- ✓ Connect the patient cable ~50cm distant to treatment location
- ✓ Before start, make sure there is a drop at the end of the cannula; wipe off the drop prior to starting the application
- ✓ When application is complete – relieve pressure on the ampule - clean the tip of the cannula and take the ampule out of the Spincare; make sure the Spincare metal tip is clean – with no residues of solution
- ✓ Discard of the ampule – remember it is for single patient/single use



## Treatment Tips

- ✓ Ensure the Spincare matrix covers the wound and frame by about 2-3 cm.
- ✓ Keep the “treatment surrounding” area dry especially around the wound.
- ✓ The Spincare matrix will become transparent allowing wound assessment.
- ✓ Secondary dressing is not required; it is recommended only during the exuding phase.
- ✓ Patient can shower regularly 24-48 hours post-treatment.
- ✓ Spincare matrix should not be removed until complete healing; unless indicated by wound condition.
- ✓ If there is colored secretion (and no other infection signs) on the Spincare matrix - wash with a sterile solution but do not remove the matrix.
- ✓ If there is a significant accumulated secretion underneath the matrix – wash the wound and re-apply Spincare matrix if required
- ✓ (If needed) Moisturize on top of the Spincare matrix.
- ✓ Allow the matrix to peel off by itself