880 mm

300 mm

## EN

### INSTRUCTION FOR USE

BWISE Silicone Foam Dressing Border

**Description**  
BWISE Silicone Foam Dressing Border has an advanced five-layer structure which consists of a soft silicone wound contact layer, a super absorbent polyacrylate fiber layer, a non-woven distributing layer, a polyurethane film layer, and a vapour permeable and waterproof polyurethane film layer.

BWISE Silicone Foam Dressing Border is highly absorbent dressing that absorbs exudates and maintains a moist wound environment.

Dressing material content:  
Silicone, polyurethane, polyacrylate, fluff pulp and polyolefin.

**Indications**  
BWISE silicone foam dressing with border is designed for a wide range of exuding wounds such as pressure ulcers, leg ulcers, venous ulcers, traumatic wounds, surgical wounds and skin tears.

The dressing may also be used as part of a prophylactic therapy to help prevent pressure ulcers.

**Instruction For Use**

BWISE Silicone Foam Dressing Border may be used by lay person under supervision of health care professionals.

1. Cleanse the wound according to clinical practice. Dry the surrounding skin thoroughly.

2. Select an appropriate size/shape. The wound pad should cover the dry surrounding skin by at least 1-2 cm.

3. Remove the first release film and apply the adhesive side to the wound.

4. Remove the remaining release film(s) and smooth down the border on the skin. Do not stretch the dressing.

5. The dressing change interval may be several days, or change the dressing before it is fully saturated, at signs of leakage, or as indicated by clinical practice. The dressing can be used under compression bandaging.

**Precautions and warning**

1. It should not be used on persons with a known allergy or hypersensitivity to the dressing or its components.

2. Do not use the product together with oxidizing agents such as hypochlorite solutions or hydrogen peroxide.

3. If you see signs of clinical infection e.g. fever or the wound becoming red, warm or swollen, consult a health care professional for appropriate treatment.

4. Do not reuse. If reused, performance of the product may not meet expectations and cross contamination may occur.

5. Sterile. Do not use if sterile packaging is damaged or opened prior to use. Do not reuse.

6. In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary on these population groups, this dressing should be used with caution following a clinician's recommendation.

**Contraindications**

None.

**Storage and disposal**

1. Keep in a clean and dry place in specified temperature range and away from sunlight.

2. Disposal should be handled according to local environmental procedures.

**Other information**

If any serious incident has occurred in relation to the use of BWISE Silicone Foam Dressing Border, it should be reported to the manufacturer and competent authority of the Member States.

**Shelf life:** 3 years from date of manufacture

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