880 mm

EN

**INSTRUCTION FOR USE**  
**BWISE Super Absorbent Dressing Adherent**

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**Indicaciones**  
BMSE Super absorbent dressing is intended for used on moderate to heavily exuding wounds, such as: leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds.

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1. After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to the wound edge.

2. BWISE Super absorbent dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, BMSE Super absorbent dressing will be in close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.

3. The frequency of changing dressing depends on the amount of exudates.

4. Dressing removal can be facilitated by moistening the dressing with saline solution.

**Precautions and warning**

1. The product is single use. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance, and Cross-contamination may occur.

2. Sterile. Do not use if sterile packaging is damaged or opened prior to use. Do not re-sterilize.

3. Do not cut the dressing, the cover must remain intact.

4. In the absence of available data supporting the use of this dressing, sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

**Contraindications**

1. Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressings swell during absorption.

2. Do not use on third-degree burns.

3. BWISE Super absorbent dressing should not be used on persons with known allergy or hypersensitivity to any of its components.

**Storage**  
Keep in a clean and dry place in specified temperature range and away from sunlight;

**Disposal**  
Dispose should be handled according to local environmental procedures.

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**Shelf life:** 3 years from date of manufacture

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