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DATE/NAME

Packungsbeilage_P3_WG_10_2508

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15. August 2025, 11:14

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EN

Product description

Primary wound dressing (spray) for acute and chronic wounds. **WOUND** contains St. John's wort oil (Oleum Hyperici), Neem oil (Oleum Azadirachtae), propane, butane.

WOUND is 100% natural.

WOUND maintains a moist wound environment and enables the body to activate physiological wound healing.

WOUND protects the surrounding skin against maceration and supports healthy skin function.

WOUND prevents the secondary dressing from adhering to the wound, thereby reducing skin trauma and pain during dressing change.

WOUND has an antimicrobial effect based on a physical mode of action.

WOUND is not irritating.

Instructions for use

Adhere to local hygiene measures and wound cleansing techniques according to standard medical practice prior to the application of **WOUND**.

1. Shake well before use.
2. Spray **WOUND** directly on the wound bed and the surrounding skin area from a distance of 5–10 cm. Ensure that the wound and surrounding skin are covered with a clearly visible oil layer.
3. Cover the wound with a suitable secondary dressing (e.g. with a non-woven gauze or absorber adjusted to the wound exudate).
4. A minimal amount of oil can remain on the spray nozzle after application. This does not affect the spray usability. If desired, clean the nozzle.

Apply more oil if the secondary dressing adheres to the wound. It is recommended to repeat the treatment every 24 to 48 hours.

Contraindications

- Do not use on individuals who have had an allergic reaction to any of the ingredients of **WOUND**.

The symbol indicates the total capacity of the aerosol container of 25 ml/75 ml.

Warning: Flammable aerosol. Pressurised container: May burst if heated. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Do not spray on an open flame or other ignition source. Keep away from heat, sparks, open flames, hot surfaces, ignition sources. No smoking. Keep out of the reach of children.

Precautions

- For external use only.
- Adjust number of puffs to wound size. Do not apply more than 15 puffs per day. For children from 1 month to 2 years of age do not apply more than 7 puffs per day.
- Since no data on the safe use in neonates and in pregnant and nursing women is available, the use of **WOUND** in these patient groups is not recommended.
- Avoid contact with eyes and mucous membranes.
- Avoid direct contact between the wound and the spray nozzle.
- Do not inhale the spray mist.
- After repeated use for 30 days interrupt the treatment for 2 days.
- In case of adverse effects such as skin irritation or allergic reactions, remove the dressing and rinse off the oil layer with saline solution or water.
- Exposure of the treated area to sunlight can trigger a phototoxic reaction if no secondary dressing is used.
- Consult a health care professional if the wound condition does not improve.
- A potential interaction between **WOUND** and other topical or systemic treatments has not been assessed.

Storage and disposal

Store **WOUND** at room temperature (15–25°C). Dispose according to local environmental regulations. Do not use **WOUND** after the expiry date marked with on the aerosol can and package.

ADVERSE REACTION / SERIOUS INCIDENT

Any adverse reaction or a serious incident suspected from this device should be reported to adversereactions@kerecis.com. Please include at least a contact phone number, email address, the device lot number and expiration date.

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Kerecis Switzerland AG, c/o
Pestalozzi Rechtsanwälte AG,
Kerecis Ltd,